

AEROHAZARD

**BODY FLUID
DISPOSAL KIT**

SINGLE APPLICATION


**A PROCEDURE FOR THE SAFE
REMOVAL AND DISINFECTION
OF BLOOD, VOMIT OR URINE.**

SEE OVER FOR INSTRUCTIONS

AERO HEALTHCARE
WARWICK HOUSE, UNIT 6 REDKILN CLOSE, HORSHAM RH13 5QL


INSTRUCTIONS FOR USE

1. Put on protective clothing provided.

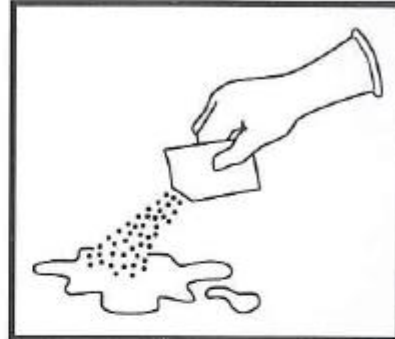

2. Sprinkle the contents of the sachet over the spill. This will solidify a liquid in 2 minutes.


3. Using the scoop and scraper provided remove the now solidified spillage and place in the Biohazard Bag.

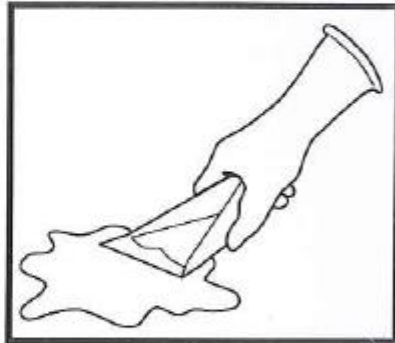

4. Use the spray to disinfect the area of the spill and wipe down with dry cloth (if included).

5. Place the used scoop, scraper and gloves in the waste bag and tie securely.

6. Wipe hands with disinfectant wipe (if included) and dispose of.

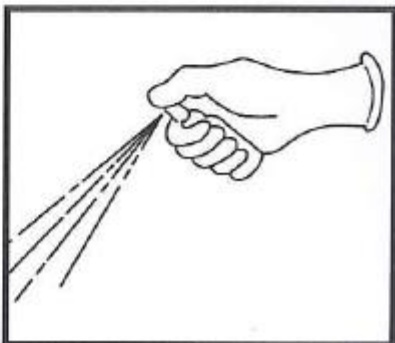

7. Dispose of the contaminated materials by approved means.